



5D LED MAGNIFYING LAMP

USER'S MANUAL

1 – INSTRUCTION FOR USE

2 –SPARE PARTS

VERRE & QUARTZ *Technologies*

Tel.: +33 1 64 66 20 33– Fax: +33 1 64 77 58 97

Email: info@verre-et-quartz.fr – Web: www.verre-et-quartz.com

Espace Vinci, 24 Avenue Graham Bell, 77600 Bussy St Georges France



1 – INSTRUCTIONS FOR USE

Dear Customer,

We congratulate you for the purchase of your VERRE & QUARTZ equipment, which will give you great satisfaction. In order to guaranty a good use, we recommend you to read carefully the principles for set-up and use below:

A. SET-UP

- If your lamp is equipped with a wall bracket, fix the support on the wall and insert the arm into the wall bracket.
- If your lamp is equipped with a rolling base, assemble as follow:
Insert the tube into the rolling base, position the washer and tighten the whole with the screw positioned on the tube.

B. PUTTING INTO SERVICE

- Before plugging the lamp, check if voltage indicated on the label and fixed to each lamp is conform with the voltage of the installation. Since the lamp is class I, you must connect it to a plug equipped with earth.
- Connect the mains wire on a 230Vac/50Hz standardised 10/16A plug.
-
- All our lamps are equipped with an ON/OFF switch, which enables to switch on or off the lamp.
- Avoid any direct exposure of the lens to the sun rays

C. TECHNICAL CHARACTERISTICS

- | | | | |
|---------|--------------------|---------------|-------------|
| - Power | 6 W | - Voltage | 230Vac 50Hz |
| -LED | 64 LED | - Arm range | 860 mm |
| - Lens | 145 mm diameter, | - magnifying: | 5 diopters |
| -On/Off | Switch on the head | - colour: | white |

D. USE

- The 5D LED lamp is specifically designed for dermatologists, aestheticians, with homogeneous lighting on the observed surface.

E. MAINTENANCE

- Unplug the lamp from the mains before any intervention.
- The lamp is “IP 20”: don’t switch it on if there is humidity in the room or a risk of corrosion
- This product cannot be modified. Any modification of this lamp can make the product dangerous



F. WARRANTY

Your lamp is 2 years pieces and labour in our assembly shop within the framework of a normal use and except from wear pieces and bulbs.

2-SPARE PARTS

Listing of the main spare parts

Designation

Stainless tube 950 mm diam 32, cap 13 mm
Rolling stand with 5 castors VQG Heavy
table bracket
wall bracket

Reference

DMK017
DMK005
DMK002
DMK003