

MAGNIFYING LAMP VISTA HANDLE

USER MANUAL

1 – INFORMATION

2 – PRECAUTION OF USE



VERRE & QUARTZ Technologies

Tel.: +33 1 64 66 20 33– Fax: +33 1 64 77 58 97

Email: info@verre-et-quartz.fr – Web: www.verre-et-quartz.com

Espace Vinci, 24 Avenue Graham Bell, 77600 Bussy St Georges France



1 – INFORMATION

Dear Customer,

We congratulate you for the purchase of your VERRE & QUARTZ equipment, which will give you great satisfaction. In order to guaranty a good use, we recommend you to read carefully the principles for set-up and use below:

2 – PRECAUTIONS FOR USE

HOW TO USE YOUR APPLIANCE

Before the assembly and the connection of the appliance to the sector, please check that the tension mentioned on the sticker corresponde to the sector tension. Don't touch the bulb when the appliance is ON, or when is cut since a little time.


Some types of bulb, like the halogens or the iodures, keep a high temperature, even after be turn off since some few minutes. Indeed, please wait that it becomes cold before touch them. Before replace the bulb, unplug the appliance, or in case of ceiling lamp cut the alimentation of the lamp with the light switch.

Don't use the bulb with more power that it is writing on the appliance. Without other specifications, all the other appliance are considered with a protection IP20, and don't be use on a wet or corrosive environnement. The appliance don't have to be change, and the potential interventions, should be doing by qualified persons.

All replace part should be origin manufacturing. Any change would invalidate approval of devices, and the security will be not warranty.

In case of multiple devices, wall or ceiling, connect all the fils yellow/green betwenn them and connect them to the terminal of general earth.

the symbol represente the minimum of the distance of security betwenn the bulb and the lighting object.

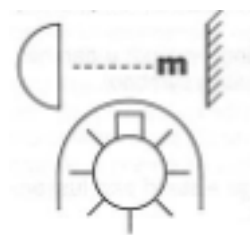

The symbol wich appears on somes lamps indicate the mandatory use of a bulb, be sure that the above symbol appears on the packaging of the new bulb.

If the cable or the extern wire of this lighting is damaged, it have to be replace by the manufacturer, his representant or all qualified persons to avoid the danger.

A. TECHNICALS CHARACTERISTICS

- Power	9 W	- Alimentation	220-230V 50-60Hz
- Life expectancy	+/- 4 000 hours	- Warranty	2 years
- Range	70 cm	- Magnification	3 diopters
- On/off switch	Interrupter	- Color	Blanc





B. USING

This medical lighting is destined to dermatology, aesthetics, precision work, surgical implant, research laboratory, veterinary medicine, dental examinations, watches, and restoration of works of art. It gives an even light.

C. WARRANTY

Your lamp is warranted for two years parts and labor in our workshops under normal use and excluding wear parts and bulbs.

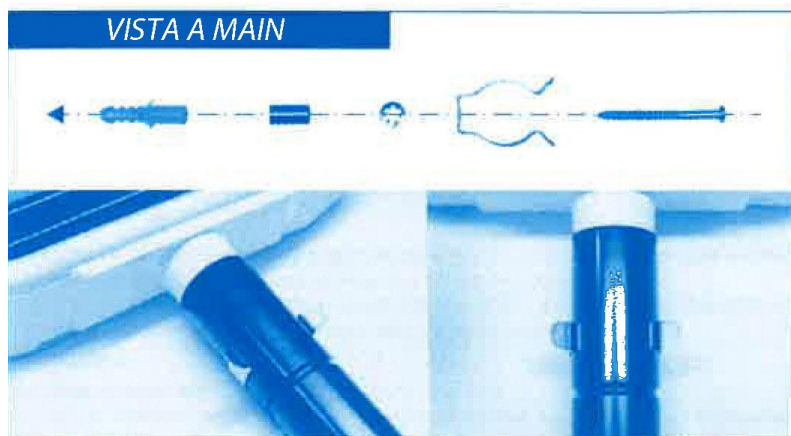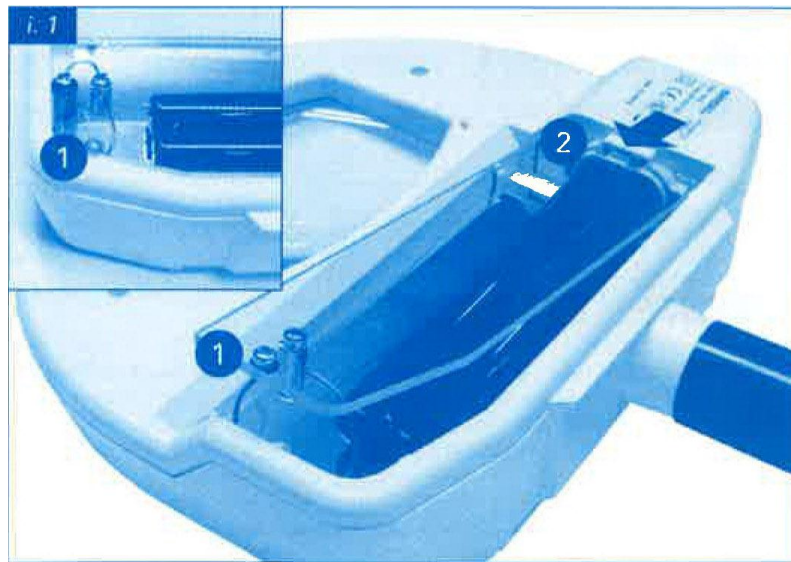

VERRE & QUARTZ Technologies

Tel.: +33 1 64 66 20 33– Fax: +33 1 64 77 58 97

Email: info@verre-et-quartz.fr – Web: www.verre-et-quartz.com

Espace Vinci, 24 Avenue Graham Bell, 77600 Bussy St Georges France