



# **VISTA MAGNIFYING LAMP**

## **USER'S MANUAL**

1 – INSTRUCTION FOR USE

2 – SPARE PARTS

**VERRE & QUARTZ *Technologies***

Tel.: +33 1 64 66 20 33– Fax: +33 1 64 77 58 97

Email: [info@verre-et-quartz.fr](mailto:info@verre-et-quartz.fr) – Web: [www.verre-et-quartz.com](http://www.verre-et-quartz.com)

24, avenue Graham Bell, espace Vinci, 77600 Bussy Saint Georges – France



## **1 – INSTRUCTIONS FOR USE**

### **Dear Customer,**

We congratulate you for the purchase of your VERRE & QUARTZ equipment, which will give you great satisfaction. In order to guaranty a good use, we recommend you to read carefully the principles for set-up and use below:

#### **A. SET-UP**

- If your lamp is equipped with a wall bracket, fix the support on the wall and insert the arm into the wall bracket.
- If your lamp is equipped with a rolling base, assemble as follow:  
Insert the tube into the rolling base, position the washer and tighten the whole with the screw positioned on the tube.

#### **B. PUTTING INTO SERVICE**

- Before plugging the lamp, check if voltage indicated on the label and fixed to each lamp is conform with the voltage of the installation. Since the lamp is class I, you must connect it to a plug equipped with earth.
- Connect the mains wire on a 230Vac/50Hz standardised 10/16A plug.
- 
- All our lamps are equipped with an ON/OFF switch, which enables to switch on or off the lamp.
- Avoid any direct exposure of the lens to the sun rays

#### **C. TECHNICAL CHARACTERISTICS**

- |         |                    |               |             |
|---------|--------------------|---------------|-------------|
| - Power | 9 W                | - Voltage     | 230Vac 50Hz |
| -Tube   | 1 x 9 W neon tube  | - Arm range   | 900 mm      |
| - Lens  | 106 x 61 mm ,      | - magnifying: | 3 Dioptries |
| -On/Off | Switch on the head | - colour:     | white       |

#### **D. USE**

- The Vista lamp is specifically designed for dermatologists, aestheticians, with homogeneous lighting on the observed surface.



#### E. MAINTENANCE

- Unplug the lamp from the mains before any intervention.
- Use a neon tube 9W when replacing it.
- The lamp is "IP 20": don't switch it on if there is humidity in the room or a risk of corrosion
- This product cannot be modified. Any modification of this lamp can make the product dangerous

#### F. WARRANTY

Your lamp is guaranteed 18 months pieces and labour in our assembly shop within the framework of a normal use and except from wear pieces and bulbs.

## **2- SPARE PARTS**

### **Listing of the main spare parts**

#### **Designation**

Stainless tube 950 mm diam 32, cap 13 mm  
Rolling stand with 5 castors VQG  
9 W fluorescent circline tube  
table bracket  
wall bracket

#### **Reference**

DMK017  
DMK006  
LTD006  
DMK002  
DMK003